INSTRUCTION FOR USE

Maternity Support Belt

- (EN) Maternity Support Belt
- (DE) Schwangerschaftsgurt
- (FR) Ceinture abdominale de maternité
- (ES) Cinturón de sujeción para embarazadas
- (SV) Graviditetsbälte
- (NO) Magestøtte for gravide
- (DA) Svangerskabsbælte

- (NL) Ondersteunende zwangerschapsband
- (PT) Cinta de apoio para maternidade
- (PL) Podtrzymujący pas ciążowy
- (HU) Terhességi hastámasztó öv
- (IT) Cintura di supporto per gravidanza
- (FI) Äitiystukivyö

- (RO) Centură de susținere pentru sarcină
- (SL) Nosečniški pas
- (RU) Дородовой пояс
- (IS) Meðgöngubelti
- (AR) حزام الدعم للحوامل
- (TR) Gebelik Destek Kemerî
- (HE) חגורת תמיכה להריון

FIBER CONTENT

(EN)	FIBER CONTENT	(HU)	ROSTTARTALOM
(DE)	MATERIALZUSAMMENSETZUNG	(IT)	COMPOSIZIONE DEL MATERIALE
(FR)	COMPOSITION	(FI)	KUITUSISÄLTÖ
(ES)	CONTENIDO DE FIBRA	(RO)	CONTIN FIBRE
(SV)	FIBERINNEHÅLL	(SL)	VSEBNOST VLAKEN
(NO)	FIBERINNHOLD	(RU)	СОСТАВ ВОЛОКОН
(DA)	FIBERINDHOLD	(IS)	TREFJAINNIGHALD
(NL)	VEZELGEHALTE	(AR)	محتوى النسيج
(PT)	TEOR DE FIBRA	(TR)	FIBER İÇERİKLİ
(PL)	ZAWARTOŚĆ WŁÓKNIEN	(HE)	תכולת הסיב

(EN)	Fabric: Polyamide, Elastane, Polyester Stays: Aluminum Foam: Polyurethane	(HU)	Szövet: poliamid, elasztán, poliészter Merevítő: alumínium Habréteg: poliuretán
(DE)	Stoff: Polyamide, Elastane, Polyester Stäbe: Aluminium Schaum: Polyurethan	(IT)	Tessuto: poliammide, elastane, poliestere Dispositivi di fissaggio: alluminio Schiuma: poliuretano
(FR)	Textile : polyamide, élasthanne, polyester Renforts : aluminium Mousse : polyuréthane	(FI)	Kangas: Polyamidi, elastaani, polyesteri Tuet: Alumiini Vaahto: Polyuretaani
(ES)	Tela: poliamida, elastano, poliéster Soportes: aluminio Espuma: poliuretano	(RO)	Tesătură: poliamidă, elastan, poliester Stabilizatoare: aluminiu Spumă: poliuretan
(SV)	Tyg: polyamid, elastan, polyester Stag: aluminium Skum: polyuretan	(SL)	Tkanina: poliamid, elastan, poliester Oporni elementi: aluminij Pena: poliuretan
(NO)	Stoff: Polyamid, Elastan, Polyester Korsettstenger: Aluminium Skum: Polyuretan	(RU)	Ткань: полиамид, эластан, полиэстер Стрекни: алюминий Пена: полиуретан
(DA)	Tekstil: polyamid, elastan, polyester Stanger: aluminium Skum: polyurethan	(IS)	Efni: Pólýamíð, elastan, pólýester Stangir: Ál Svampur: Pólýúretan
(NL)	Stof: polyamide, elastaan, polyester Stangen: aluminium Schuim: polyurethaan	(AR)	النسيج: بولي أميد، إيلاستان، بوليستر الدعامات: الومينيوم الإسفنج: بولي يوريثان
(PT)	Tecido: poliamida, elastano, poliéster Suportes: alumínio Espuma: poliuretano	(TR)	Kumaş: Poliamid, Elastan, Polyester Çubuklar: Alüminyum Köpük: Poliüretan
(PL)	Tkanina: poliamid, elastan, poliester Usztywniacze: aluminium Pianka: poliuretan	(HE)	בז: פוליאםיד, אלסטין (סידקס), פוליאסטר חיזוקים: אלומיניום פונוג: פוליאורטן

JOBST PRODUCT FEATURES

(EN) JOBST PRODUCT FEATURES

(DE) JOBST PRODUKTEIGENSCHAFTEN

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EN

Dear Customer,

thank you for choosing the JOBST® Maternity Support Belt.

The JOBST® Maternity Support Belt is a back support for pregnant women. It is designed to comfortably support and relieve the lumbar and abdominal area, thus helping to ease back pain as well as other ailments in the lower back area during pregnancy.

The JOBST® Maternity Support Belt can be adjusted to increasing abdominal girth during the various stages of pregnancy. The two-stage hook-and-loop fastener system ensures simple handling and adjustment. Initially, a wider main belt embraces the abdomen. Then additional elastic side straps provide added support if needed and allow for individual adjustment. The multi-layer laminate ensures firm and comfortable support while two light, individually formable rods in the back area further stabilize and prevent the belt from curling. The JOBST® Maternity Support Belt can be worn discreetly and comfortably underneath almost any garment.

Please follow the instructions below.

Indications:

The JOBST® Maternity Support Belt can be used

- For lower back pain during pregnancy
- For degenerative changes in the lumbar spine region
- For instability in the lumbar spine region
- To prevent over-stressing of tendons, muscles and ligaments

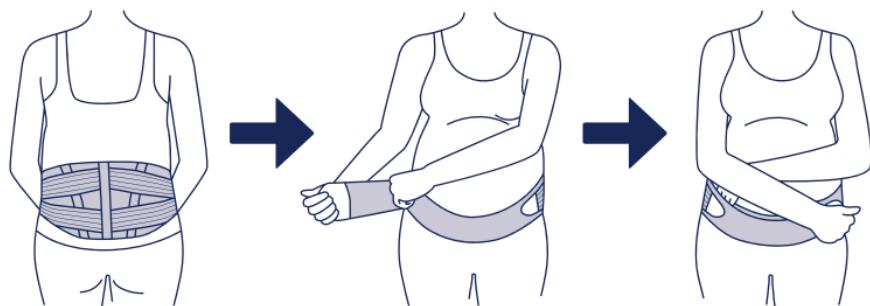
We recommend wearing the JOBST® Maternity Support Belt over a light top. Wash if required.

Contraindications:

Do not use the JOBST® Maternity Support Belt in case of serious back injury and known allergies or hypersensitivity to any of the components.

Application:

The JOBST® Maternity Support Belt is available in four different sizes graduated to also cover any intermediate sizes. We recommend fitting and adjustment of the appropriate size by a medical professional to ensure proper fit and therapeutic support. The belt should be put on while standing. A larger mirror may help when putting on the belt for the first time.

INSTRUCTIONS FOR USE

EN

1. Open the support belt. Before fitting make sure that the label "Top" faces upwards.
2. Position the support belt in the center of your lower back, then grasp both ends and pull to the front. The thumb loop on the left facilitates application of the support belt.
3. Wrap the slightly tensed support belt around the lower abdomen and secure it using the hook-and-loop fasteners. Adjust the main belt's closure until you feel comfortable support in the lower back and abdomen region.
4. The elastic side straps located on the outside of the main belt allow further adjustment of the stability level. To do so, pull both straps simultaneously and secure the hook-and-loop fasteners on the main belt in a comfortable position.
5. If necessary, both stabilizing elements at the back may be shaped to fit individually.
6. Avoid any creases in the clothing underneath the belt.
7. Fasteners may be adjusted individually depending on the situation and personal preference (e.g. when sitting down).

After taking off the support belt or before re-application, the outer side straps should be attached loosely in their original position.

Care and cleaning:

Close all fasteners before washing. Machine wash at 30 ° C / 86 ° F (delicate cycle) or hand wash in lukewarm water using mild detergent. Do not bleach, tumble dry, iron or dry clean belt. Dry at room temperature and make sure the belt is completely dry before re-applying.

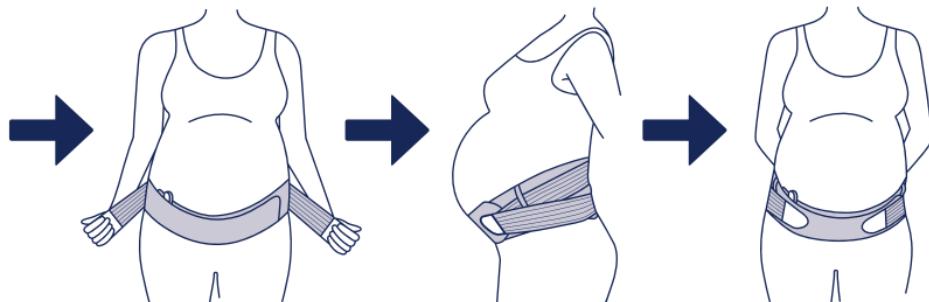
Precautions:

Please follow your physician's therapy recommendations. Should you experience any unusual swelling, numbness, skin discoloration or discomfort, stop wearing the JOBST® Maternity Support Belt and consult a medical professional.

Any serious incident that has occurred in relation to this device should be reported to the BSN medical Inc. and the competent authority of your state.

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BSN-Jobst GmbH
EC REP BSN medical GmbH